

CT-12120D DIGITAL THAW UNIT



Thank you for selecting the CT-12120D Digital Thaw Unit.

This electronic device automatically heats and maintains an accurate water temperature. This Digital Thaw Unit was designed to give you years of trouble free service.

INCLUDED IN KIT



Lift Out Basket

TO OPERATE

1. Fill Digital Thaw Unit with clean tap water. Plug cord into correct voltage source as indicated on base of unit. Wait for thaw device to heat up to proper temperature. Your water temperature when you started determines the exact warm up time approximately 2°F/1.1°C per minute.

50° water temp. to 96.5°F/36°C = approx. 23 min.
60° water temp. to 96.5°F/36°C = approx. 18 min.
70° water temp. to 96.5°F/36°C = approx. 13 min.
80° water temp. to 96.5°F/36°C = approx. 8 min.

2. Green ready light must be on, indicating unit is up to temperature.

3. Agitate the water by taking the basket and moving it up and down several times to assure water temperature is consistent through the water bath.

PROCEDURE TO CHECK WATER TEMPERATURE

4. With the correct water temperature you are now ready to thaw semen in your Thaw Unit.

5. *NOTE:* New Thaw Units have an internal temperature adjustment knob. Your Digital Thaw Unit has been factory calibrated to 96.5°F/36°C. However, if you desire to change the preset temperature, you may readjust it by removing the 7 screws on the bottom of the unit and adjusting the knob clockwise for warmer and counter-clockwise for cooler temperatures. Turn temperature adjustment knob a little at a time and allow unit to stabilize before checking temperature again.

ALSO AVAILABLE
in 220 vac. Please inquire.

Do not use with a portable generator.

Keep electrical connections
clean and dry.

TWO YEAR WARRANTY



The Coburn Company, Inc. PO Box 147 Whitewater, WI 53190 U.S.A.
800 776 7042 262 473 2822 coburn.com info@coburn.com